

Perfectín

Compuesto para Modelar

Tipo 1 Godiva para Impresiones

La Godiva Perfectín es un material destinado a la toma de impresiones de maxilares total o parcialmente desdentados.

Precauciones:

- No use este producto después de la fecha de caducidad.
- USO EXCLUSIVO A PROFESIONALES INSTITUCIONES SANITARIAS.

Instrucciones:

La Godiva Perfectín se calienta en agua alrededor de 60°C (se recomienda el uso de un termostato) hasta que se ablande completamente. Luego se la amasa con los dedos manteniéndola ligeramente por encima de la superficie del agua. Una vez cargada la cubeta se glasea la superficie sobre una llama

y se atempera sumergiéndola nuevamente en el agua caliente. La impresión se toma según la técnica habitual.

El endurecimiento puede acelerarse refrigerando con aire y/o agua. Si hiciera falta alguna corrección se emplea Compuesto para Modelar Perfectín en lápices.

Si bien este material no requiere vaciado inmediato, cuanto antes se haga mejores serán los resultados.

Condiciones de Almacenamiento y Conservación:

Conserve en el lugar fresco y seco.

Presentación:

8 tabletas.

Perfectín Dental Impression

Compound - Type 1

The Godiva Perfectín is a material intended to be used for taking impressions of

totally or partially edentulous jaws.

Precautions:

- Do not use this product after the expiration date.
- EXCLUSIVE USE FOR HEALTH PROFESSIONALS AND INSTITUTIONS.

Instructions:

Godiva Perfectín is heated in water around 60 °C (the use of a thermostat is recommended) until it is completely softened. Then you knead it with your fingers, keeping it slightly above the surface of the water.

Once the bucket is loaded, the surface is glazed over a flame and tempered by submerging it again in the hot water. The impression is taken according to the usual technique.

Hardening can be accelerated by cooling with air and/or water. If any correction is needed, use Perfectín Modeling Compound in pencils.

Although this material does not require immediate pouring, the sooner it is done, the better the results will be.

Storage conditions and conservation:

Store in a cool, dry place.

Presentation:

8 tablets.

**Autorizado por la
ANMAT PM 1691 - 35**

Director Técnico:

Leandro A. Liria.

MP 16212. Farmacéutico
970502 Rev 02O - Fecha de
publicación / Publication date
(2025-02)



LABORATORIOS SL S.A.

Curupaytí 2611- B1644GDC -
San Fernando Argentina -
www.subiton.com