Pickav

Material radiopaco a base de hidroxido de calcio

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Pickav

Radiopaque calcium hydroxide base material

Indications

Pickav is a cement that sets to a solid mass that can be used for pulpal vitality protection under every type of restoration. This effect is due to the calcium hydroxide that creates an environment that favors dentin

calcification and inhibits bacterial growth.

Precautions:

- Do not use this product after the expiry date.
- EXCLUSIVE USE FOR HEALTH PROFESSIONALS AND INSTITUTIONS.

Instructions

Use approximately equal amounts of base paste and catalyst paste (1.16:1.00 by weight) dispensing them on a paper pad.

It is convenient to prepare the mix with a small spherical-ended instrument that can be also used to place the material on the tooth areas where it is needed.

The total mixing time need not exceed 40 seconds for the quantities normally used. Setting occurs between 1'30' to 5 minutes at 23°C, and 40-60 % humidity. It should be noted that setting may occur more rapidly in the mouth, or in conditions of higher temperature and, especially, higher humidity. The mechanical strength of Pickav allows for condensation of the amalgam. However, several studies have established that it is not advisable to leave the restoration resting on cements of this type.

The storage of the product at high temperatures can lead to separation of the solid and liquid phases inside the tubes.

Package

1 tube base paste 13 g
1 tube catalyst paste 11 g

Storage and Conservation:

Store in dry and cool place with well sealed containers.

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LABORATORIOS SL S.A.