Prothoquick

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Prothoquick

This acrylic resin for denture bases, is easy to handle and fast curing both in hot water and microwave oven

Indications

This acrylic resin for denture bases, is easy to handle and fast curing both in hot water and microwave oven.

Characteristics:

- It polymerizes in 15 minutes in boiling water.
- It polymerizes in 6 minutes in microwave oven. It does not require pressure control.
- It has a higher elasticity than conventional acrylics.
- It provides a greater thermal and dimensional stability.

Warnings and precautions:

- Use the liquid in a well-ventilated area. Cover it when it is not in use.
- Do not use this product after the expiration date.
- Avoid skin contact with the monomer and unpolymerized material.

Conventionalcuring - Instructions for use:

Only use prothoquick liquid, and rotate the flask before using it, to obtain the best product properties. It is recommended to use the proportion of 2,4 g of powder for each milliliter of liquid (Powder: Liquid ratio of 24g:10mL).

If there are not the necessary elements for measurement, pour approximately 5 or 10 mL of ProthoQuick Liquid, as required, into a mixing container and spread the necessary amount of powder onto the liquid (until saturation). Gently tap the bottom of the container 2 or 3 times and invert the bowl to eliminate the excess powder. Mix for about 20 seconds and cover.

The ProthoQuick acrylic reaches the dough like consistency required for packing 4 - 6 minutes after the beginning of the mixing procedure, and remains in this state for another 15-25 minutes (depending on the ambient temperature). The dough can be used at any moment within this period.

Pack the material into the mold and cover it with a polyethylene sheet. Place the counter-mold and press slowly by means of a press. Open the mold and remove the polyethylene sheet, trim the overflowing material around the mold with a sharp instrument (better results can be obtained if a polyethylene sheet is once again used for pressing). Place the counter-mold and press until it completely closes. Then transfer the mold into a muffle clamp, before curing.

Curing

One of the main advantages of the ProthoQuick acrylic is its outstanding polymerization performance. Immerse the muffle/s with the corresponding clamp/s directly into boiling water or, start at ambient

temperature, on a low flame, and let it boil for 15 minutes after it starts boiling again, or after it reaches the boiling temperature.

Once the polymerization cycle is completed, let it first cooldown in ambient air for 10 minutes, and then let it cool down completely in cold water.

Finish and polish in the usual way.

Microwaveoven curing - Instructions for use:

As opposed to the previous process, the powder/liquid mix will be measured: 3 powder parts per 1 liquid part. Then follow the conventional procedure.

The ProthoQuick acrylic reaches the mass consistency required for packing (between filament forming and plastic) approximately 6 minutes after the beginning of the mixing procedure. At that moment, perform the first pressing, with a polyethylene sheet. When the plastic stage is reached, between minute 7 and 10, open the mold, trim the overflowing material, place the polyethylene sheet and press. Proceed with the curing step.

Note: We suggest not to wait until the plastic cutting stage to press, because the plastic muffles far microwave oven cannot withstand excessive pressure.

Curing

The mold is placed directly onto the turning dish of the microwave oven, with the screws facing downwards. Program 3 minutes at 320 W (30/40%). Then, let it rest for 4 minutes inside the microwave oven. Again, program 3 more minutes at 720W (80/90%). Finally, remove the muffle from the microwave oven and let it rest and cool down for 10 more minutes. It can then be cooled down in water. Open the muffle when it is cold.

Note: The times and powers mentioned are based on an 800 W microwave oven.

Presentation:

Powder in 400g and 1Kg
Liquid in 170ml, 500ml and 1L

Storage conditions:

- Keep out of reach of children
- Store in a dark, cool and dry place.

EXCLUSIVE USE FOR HEALTH PROFESSIONALS AND INSTITUTIONS.

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LABORATORIOS SL S.A.